
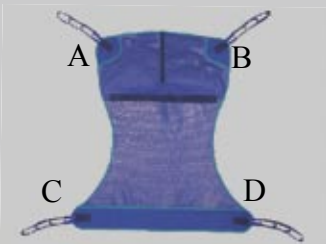


# Apexlift Sling User Guide

## INVACARE® COMPATIBLE SLINGS

For Easier Sling Applications,  
Please Consider  
APEX UNIVERSAL SLINGS

Diagram 1

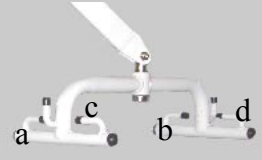


Step 1: Check sling weight capacity for the patient being lifted. Do not exceed maximum safe working load.

Step 2: Review following procedures carefully prior to attempting lift. Call with any question.

Step 3: Position the sling under and behind the back with the color adjustment strap on outside. A & B should be draped over the shoulders as in Diagram 2.

Step 4: Carefully, with help from another caregiver, if needed, pull the sling portion C & D forward to the bend of the knee as in Diagram 2.

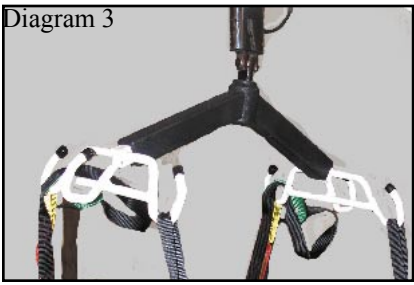

Step 5: Hook Loops A & B on sling to a & b on the spreader bar as in Diagram 3.


Step 6: Hook loops C & D on sling to c & d on spreader bar as in Diagram 3.

Step 7: Do not lock caster brakes to allow lift to properly balance weight.

Step 8: Proceed with lift and transfer as in Diagram 4.



- SLING LOOP OPTIONS**
- Long Loop- reclined position.
  - Center Loop- semi-reclined position
  - Short Loop- most upright position

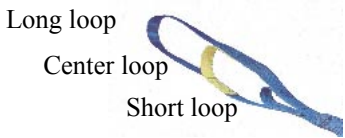

SEE REVERSE FOR ADDITIONAL SLING OPTIONS

INVACARE® is a registered  
trademark of INVACARE CORPORATION

<b>UNIVERSAL SLINGS MESH SPLIT LEG</b>	
SL-UM882	Universal mesh medium 400lbs. (DSL884)
SL-UM883	Universal mesh large 400lbs. (DSL885)

<b>UNIVERSAL SLINGS DELUXE PADDED SPLIT LEG</b>	
SL-UP892	Universal padded medium 400lbs. (DSLGU2)
SL-UP893	Universal padded large 400lbs. (DSLGU3)
SL-P894	Universal padded bariatric 600lbs. (DSLGU6)

<b>HAMMOCK SLINGS DELUXE PADDED with full head support</b>	
SL-HP222	Hammock padded medium 400lbs. (DSLGX2)
SL-HP223	Hammock padded large 400lbs. (DSLGX3)

<b>PEDIATRIC SLINGS</b>	
SL-PM111	Pediatric mesh (18"-28" torso) patient weight recommended 100lbs. or less
SL-PM121	Pediatric padded (18"-28" torso) patient weight recommended 100lbs. or less

<b>SANI SLINGS DELUXE PADDED (TOILETING)</b>	
SL-TP332	Sani padded medium 400lbs. (torso 30"-48") (DSLGC2)
SL-TP333	Sani padded large 400lbs. (torso 38"-60") (DSLGC3)

<b>STAND ASSIST SLINGS DELUXE PADDED</b>	
SL-SA662	Stand assist padded medium 400lbs.(torso 30"-48") (DLSA2)
SL-SA663	Stand assist padded large 400lbs. (torso 38"-60") (DLSA3)
SL-SA669	Buttocks support strap for stand assist (DLSA9)

<b>INVACARE REPLACEMENT SLINGS: MESH FULL BODY</b>	
SL-R110	Mesh full body medium 450lbs.
SL-R111	Mesh full body large 450lbs.
SL-R114	Mesh full body medium 450lbs. with commode opening
SL-R115	Mesh full body large 450lbs. with commode opening
SL-R116	Mesh full body extra large 450lbs. with commode opening
SL-R140	Mesh full body XX-large bariatric 600lbs.
SL-R141	Mesh full body XX-large bariatric 600lbs. with commode opening

<b>INVACARE REPLACEMENT SLINGS: SOLID FULL BODY</b>	
SL-R112	Solid fabric full body medium 450lbs.
SL-R113	Solid fabric full body large 450lbs.
SL-R117	Solid fabric full body extra large 450lbs.