
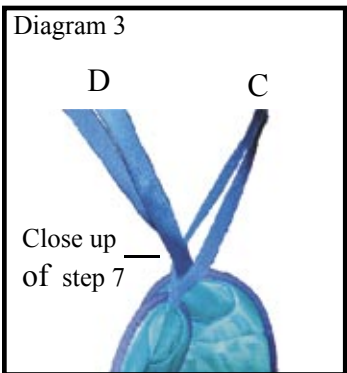


Apexlift Sling User Guide

SANI SLINGS (Toileting)

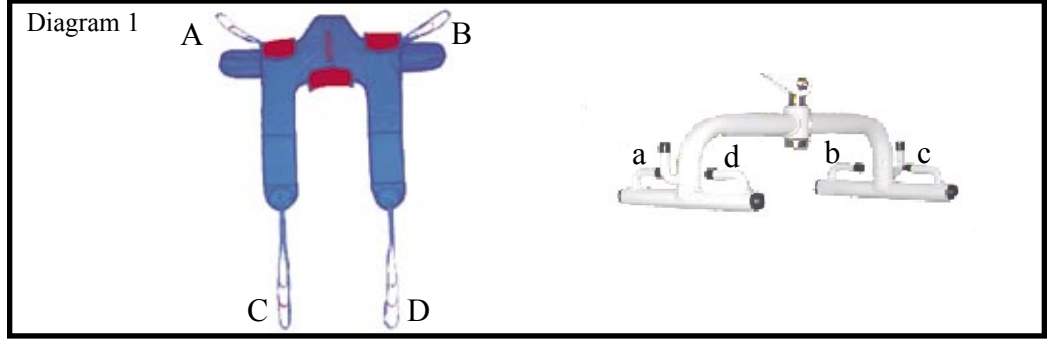

Step 1: Check sling weight capacity for the patient being lifted.

Step 2: Review following procedures carefully prior to attempting lift. Call with any questions.

Step 3: Position the sling under and behind the back with color adjustment strap outside. The two red padded sections next to A & B should be positioned under the arms (Diagram 2).

Step 4: Cross the velcro belt (E) over chest and fasten as in Diagram 2.

Step 5: Loop straps C & D around and under legs as in Diagram 2.

Step 6: Hook loops A & B to spreader bar a & b with arms outside the sling loops (Diagram 4).

Step 7: Cross over C & D placing loop C through loop D as in Diagram 3.


Step 8: Hook loops C & D to hooks c & d on spreader bar as in Diagram 4.

Step 9: Do not lock caster brakes to allow lift to properly balance weight.

Step 10: Proceed with lift and transfer to commode as in Diagram 5.

SLING LOOP OPTIONS

Long Loop- reclined position.
Center Loop- semi-reclined position
Short Loop- most upright position

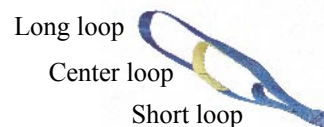

SEE REVERSE FOR ADDITIONAL SLING OPTIONS

UNIVERSAL SLINGS MESH SPLIT LEG	
SL-UM882	Universal mesh medium 400lbs. (DSL884)
SL-UM883	Universal mesh large 400lbs. (DSL885)

UNIVERSAL SLINGS DELUXE PADDED SPLIT LEG	
SL-UP892	Universal padded medium 400lbs. (DSLGU2)
SL-UP893	Universal padded large 400lbs. (DSLGU3)
SL-P894	Universal padded bariatric 600lbs. (DSLGU6)

HAMMOCK SLINGS DELUXE PADDED with full head support	
SL-HP222	Hammock padded medium 400lbs. (DSLGX2)
SL-HP223	Hammock padded large 400lbs. (DSLGX3)

PEDIATRIC SLINGS	
SL-PM111	Pediatric mesh (18"-28" torso) patient weight recommended 100lbs. or less
SL-PM121	Pediatric padded (18"-28" torso) patient weight recommended 100lbs. or less

SANI SLINGS DELUXE PADDED (TOILETING)	
SL-TP332	Sani padded medium 400lbs. (torso 30"-48") (DSLGC2)
SL-TP333	Sani padded large 400lbs. (torso 38"-60") (DSLGC3)

STAND ASSIST SLINGS DELUXE PADDED	
SL-SA662	Stand assist padded medium 400lbs.(torso 30"-48") (DLSA2)
SL-SA663	Stand assist padded large 400lbs. (torso 38"-60") (DLSA3)
SL-SA669	Buttocks support strap for stand assist (DLSA9)

INVACARE REPLACEMENT SLINGS: MESH FULL BODY	
SL-R110	Mesh full body medium 450lbs.
SL-R111	Mesh full body large 450lbs.
SL-R114	Mesh full body medium 450lbs. with commode opening
SL-R115	Mesh full body large 450lbs. with commode opening
SL-R116	Mesh full body extra large 450lbs. with commode opening
SL-R140	Mesh full body XX-large bariatric 600lbs.
SL-R141	Mesh full body XX-large bariatric 600lbs. with commode opening

INVACARE REPLACEMENT SLINGS: SOLID FULL BODY	
SL-R112	Solid fabric full body medium 450lbs.
SL-R113	Solid fabric full body large 450lbs.
SL-R117	Solid fabric full body extra large 450lbs.